Instrucciones d'installazione

(F)

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Installation instructions

(GB)

Attach the 2 x L-shaped brackets to the MOPEDIA raised toilet seat: turn the model upside down on a flat surface and insert the L-shaped brackets into the slots. Push gently on the bracket and turn the knob clockwise until the thread engages. Position the brackets leaving a greater width than that of the toilet bowl.

Lift up existing toilet seat and place the MOPEDIA seat directly onto the toilet bowl positioning the brackets towards the rear and the lip against the front of the bowl.

Turn the adjustment knobs clockwise until a resistance is felt and the seat does not move from side to side.

Ensure seat is firmly fixed before using.

ATTENTION! The side-locks have been especially designed to allow for easy tightening therefore turning should not be continued once a resistance is felt.

Cleaning Instructions

All parts can be cleaned with warm water and a neutral cleanser. Alternatively, the seat can be totally immersed in water and/or can be sanified (without brackets) in an autoclave at 85°C for 1 minute.

Cautions

Do not exceed maximum user weight: 200kg /415lb

Always ensure the toilet seat is securely fixed before using.

Do not hold onto seat for support while rising/lowering.

Technical Details

Weight capacity: 200kg/415lb

Dimensions: 40.7 x 36.6 cm. Opening 26 x 20cm.

MOPEDIA bidet

Portable bidet for use with MOPEDIA raised toilet seats. Fits snugly allowing easy transferal. Soap dish located at the front to assist personal cleaning.

User Instructions

Fill MOPEDIA bidet with warm water and place inside installed MOPEDIA raised toilet seat.

Use as normal bidet.

Tip used water into toilet bowl.

Designed and manufactured by:

Moretti S.p.A.

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)

Italy

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com