GARANZIA

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WARRANTY

This shower bench has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your shower bench repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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SHOWER BENCHES INSTRUCTION MANUAL

Moretti S.p.A.
Via Bruxelles 3 - Meieto
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

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I class medical device
according to ECC directive 93/42

Codes RS927-929-931-933

Thank you for purchasing shower bench **MOPEDIA** by Moretti, manufactured to help people with movement difficulty. It is realised in anodized aluminium tubing frame with plastic seat. It is provided with non-slip rubber tips. Adjustable height every 2,5 cm:

- **RS all models** from 37 to 47 cm

To adjust the height

Push the button to adjust the height. Draw out the stick paying attention to the corresponding holes up to the desired height which should reflect user's needs. Then leave the push button to fix the stick again. Repeat described procedure for each leg, at the end, check that all four legs have the same height. Maximum endurable weight 130 kg.

ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Cleaning

Use a damp cloth and a normal detergent. We recommend not to submerge the seat in the bath water, drain the seat tubes after use and dry the seat thoroughly. Don't use harsh abrasives or bleach based cleaners which may cause damage or discoloration.

Warning

The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the use, adjustment or maintenance of the product, please contact your dealer.
- Worn, damaged or missing components must be replaced.
- Do not store the product outdoors.
- Periodically check the stability of the product and that the screws are properly tightened and the rubber caps ends in good conditions.