SKEMA®

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I Class Medical Device according to Directive 93/42/EEC and further modifications

CODE

MO350NB	VEGA Electric examination couch, 68 cm, white
MO350NV	VEGA Electric examination couch, 68 cm, green
MO350NBLU	VEGA Electric examination couch, 68 cm, blue
MO351NBLU	VEGA Electric examination couch. 68 cm, blue, with castors
MO360NBLU	VEGA Electric examination couch, 90 cm, blue
MO361NBLU	VEGA Electric examination couch, 90 cm, blue, with castors

INTRODUCTION

Thank you for purchasing SKEMA examination couch by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives you some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

INTENDED USE

Moretti examination bed is intended for patient diagnosis, treatment and monitoring.

CAUTION!

- Do not use the product for a purpose not indicated in this manual.
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the SKEMA category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:
1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive

2. The devices in question ARE NOT MEASUREMENT DEVICES
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION

4. The devices in question are sold in NON STERILE packing
5. The devices in question should be considered as belonging to class I
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed examination bed away from heat sources
- SERVICE LIFE- the examination bed use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the bed

GENERAL DESCRIPTION

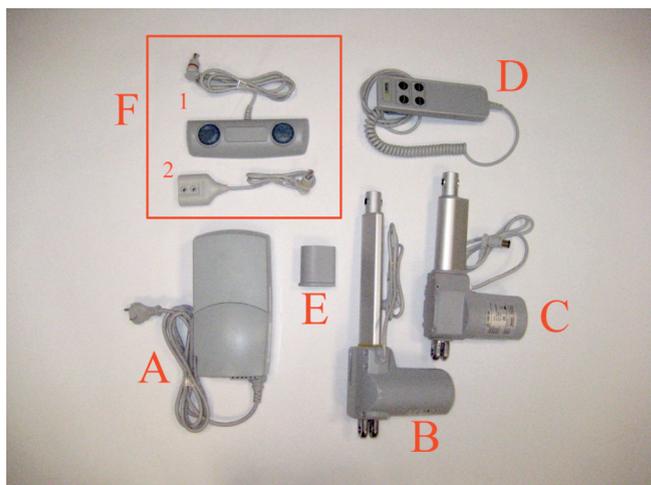
Parts view

Fig.1

Description

1. Padded headrest
2. Padded examination bed
3. Pins and ring locks
4. Footswitch with screws (only for MO351N and MO361N)

Electric kit view

Fig.2
Description:

- A. Control unit
- B. Actuator for elevation mod.TA1
- C. Head section actuator mod.TA7
- D. Handset
- E. Plastic lock
- F. OPTIONAL- footswitch MR320-321

ASSEMBLING

WARNING!

Perform these operations with an operator aid, be careful to avoid personal injuries or property damages during bed movements

Assembling the bed

1. Remove the item from its packaging and check the contents correspond to the illustrated parts description
2. Before assembling the bed is necessary to assemble the control unit (Rif. A) on the actuator for elevation (Rif. B). Insert the control unit on the actuator and block it with plastic lock (Rif. E), as shown in the sequence below (Fig.3-4-5) (pag.22)

Fig.3

Fig.4

Fig.5

3. Place the bed carefully on its side
4. Assemble the actuator and the control unit to the bed, using 2 pins and 2 ring locks (Rif. 3) as shown in Fig.6-7

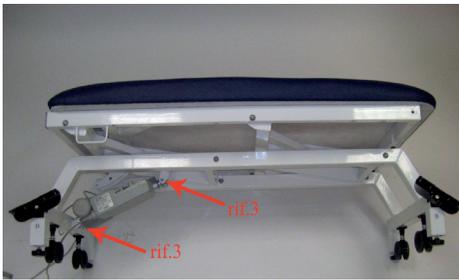
Fig.6

Fig.7

5. ONLY FOR M0351N-M0361N

Assemble the footswitch for castors, following the reference letters shown on the footswitch and on the frame, and lock them with M5 Phillips head screws (Fig.8) (Rif4)

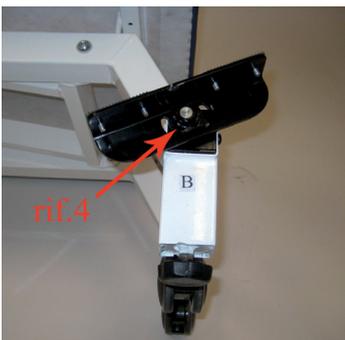
Fig.8

Follow the procedure for both sides

6. Connect the actuator and the handset as shown in (Fig.9)

- Handset connection (Rif. 1)
- Actuator for elevation connection (Rif. 2)
- Head section actuator connection (Rif. 3)

Fig.9

7. Raise the bed

8. Assemble the headrest using 2pins and 2ring locks as shown in Fig.10 (Rif3)

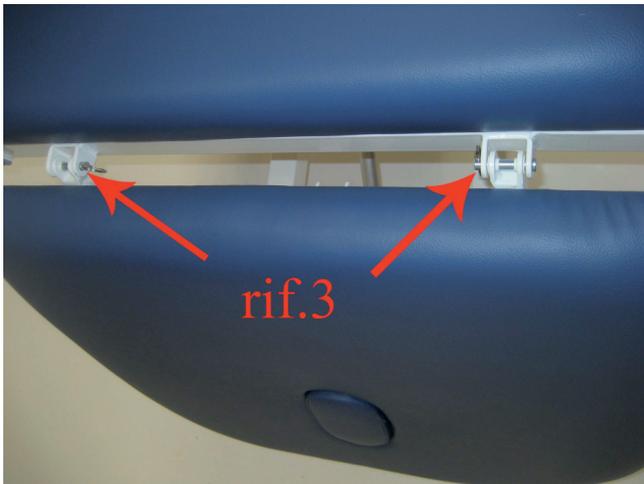
Fig.10

9. Fit the actuator for head section (C pag. 21) using 2 pins and 2 ring locks as shown in Fig.11 (Rif. 3)

Fig.11

10. Connect the actuator for head section as shown in Fig.9
11. Assure the cables to the control unit with its locks, and place the cover as shown in Fig.12

Fig.12

12. **ONLY FOR M0351N-M0361N**

Place the bed on its side. Follow the instruction on point 5 to assemble the castors

13. Raise the bed correctly

- Let the footswitch correctly works by adjusting the foot height. Do not rise more than 10mm (Fig.13)

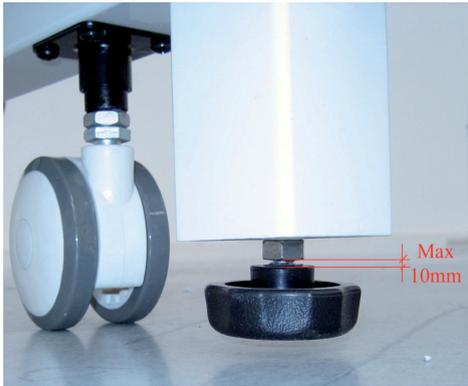
Fig.13

THE BED IS DULY ASSEMBLED

Assembling the footswitch MR320-321 (Optional)

- Remove the plastic cover on the control unit with the help of a flathead screwdriver
- Disconnect the handset electric connector (if installed), plug in the Y cable (Fig.14)
- Assure the cables to the control unit with its locks, and place the plastic cover
- Fix the Y cable on the intended leg with two holes with 2 M5 screws (Fig.14)

Fig.14

5. Plug the handset and the footswitch connectors into the Y adapter (Fig.15)

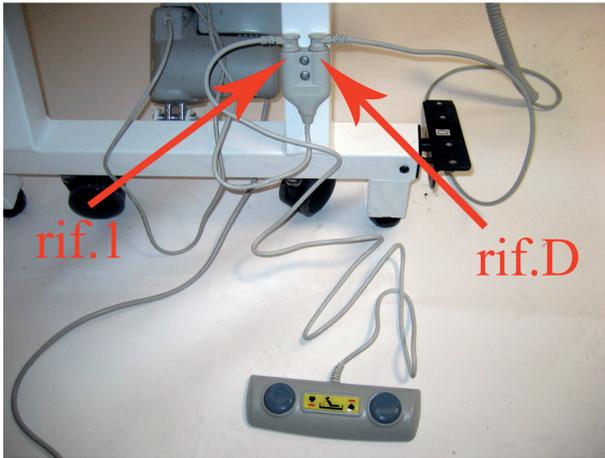
Fig.15

THE FOOTSWITCH IS CORRECTLY ASSEMBLED AND READY FOR USE

Assembling paper roll holder M0355B (Optional)

1. Fix the supports on the frame using 2 knobs as shown in Fig.16

Fig.16

2. Insert the paper roll holder and lock with the knob as shown in Fig.17

Fig.17

THE PAPER ROLL HOLDER IS CORRECTLY ASSEMBLED AND READY FOR USE

BEFORE USE

- Check if the frame is damaged, in order to guarantee a safe use for people and things
- Check the bed is correctly assembled, paying particular attention to headrest and actuator lock system

WARNINGS FOR A CORRECT USE

- Before any movements assure the bed is positioned on its feet
- Before clean, maintenance or movements disconnect the plug
- Do not use additional multiple outlets or extension cables
- Do not connect not suitable electrical devices

HOW TO USE

BEFORE USE CONNECT THE POWER SUPPLY TO THE ELECTRIC OUTLET 220 VAC

-Use the handset

- A- Raise head section
- B- Lower head section
- C- Raise bed
- D- Lower bed

NOTE The yellow Led indicates the handset is working

Use the castors (only for M0351N-M0361N)

CAUTION!

BEFORE MOVING THE BED, ALWAYS UNPLUG THE SUPPLY CABLE

The bed are equipped with 4 swivel castors.

To use the castors press the pedals at the sides of the beds (left and right), in order to lift the bed from its feet.

Move the bed to the desired position, then press the pedals again to lift the castors and let the feet lean on the floor

Use the footswitch MR320-MR321 (optional)

- MR320** footswitch for head section elevation
- MR321** footswitch for bed elevation

The two models are different only in their specific function.

1. To lift press the “up” arrow (for head section or for the bed)
2. To lower press the “down” arrow (for head section and for the bed)

NOTE follow the chapter “ASSEMBLY” to assembly the castors

MAINTENANCE

The SKEMA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market.

For patient and physician’s safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

We also recommend a periodic examination to the following parts:

- The frame and the raising mechanism, including connections, controls and safety
- Footswitch assembly and functioning
- Cables and plug integrity
- Welding points

In case of repair, use only original spare parts and accessories.

CLEANING AND DISINFECTION

Cleaning Use a damp cloth and mild soap. Then dry thoroughly before use
 water at a temperature not exceeding 30°
 Do not to use washing machines jet of water and steam

Disinfection If you need to disinfect the device use a common disinfectant cleaner

NOTE Never use acids, alkalis or solvents such as acetone

CONDITIONS OF DISPOSAL

General conditions of disposal

In case of disposal don’t use the inserting container for municipal waste. We recommend to dispose the beds in the appropriate disposal areas for recycling.

TREATMENT OF ELECTRICAL PARTS

(Directive 2002/96/EC)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the products is marked with crossed mobile waste bin

SPARE PARTS/ACCESSORIES

For spare parts and accessories refer to Moretti main catalogue.

Available accessories

MR320	Footswitch for head section elevation
MR321	Footswitch for bed elevation
MO355B	Paper roll holder

TROUBLESHOOTING

SYMPTOM	CAUSES	SOLUTION
No control running	<ol style="list-style-type: none"> 1. The plug is not connected 2. The handset is not connected to the control unit 3. The handset or the control unit are defective 	<ol style="list-style-type: none"> 1. Connect the plug 2. Check and connect the handset 3. Contact the service center for a more precise diagnosis
The head-section elevation doesn't work	<ol style="list-style-type: none"> 1. The head-section actuator is not connected to the control unit 2. The actuator or the handset are defective 	<ol style="list-style-type: none"> 1. Check and connect the elevator to the control unit 2. Contact the service center for a more precise diagnosis
The bed elevation doesn't work	<ol style="list-style-type: none"> 1. The bed actuator is not connected to the control unit 2. The actuator or the handset are defective 	<ol style="list-style-type: none"> 1. Check and connect the actuator to the control unit 2. Contact the service center for a more precise
The footswitch doesn't work	<ol style="list-style-type: none"> 1. The footswitch is not connected to the Y cable 2. The Y cable is not connected to the control unit 3. The control unit or the actuator are defective 	<ol style="list-style-type: none"> 1. Check and connect the footswitch to the Y cable 2. Connect the Y cable to the control unit 3. Contact the service center for a more precise diagnosis

TECHNICAL FEATURES

Dimension

MO350NB
MO350NV
MO350NBLU
MO351NBLU

Dimension	cm 202x68
Min. height	cm 50
Max. height	cm 90
Max head section angle	62°

MO360NBLU
MO361NBLU

Dimension	cm 202x90
Min. height	cm 50
Max. height	cm 90
Max head section angle	62°

Technical specifications

Max. weight	150 kg
Total weight with standard electric equipment	
MO350N-MO351N	60 kg
MO360N-MO361N	77 kg
Force required to operate the controls on the remote control	5N
Power supply	220 Vac 50-60 Hz
Output voltage from the control unit	24 VDC
Protection Class	IP 54
Noise level	< 45 DB
Insulation Class	2 Class
Material	Varnished steel frame, foam bed cover in PVC

WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

REPAIRING

WARRANTY REPAIR

If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

REPAIR NOT COVERED BY WARRANTY

A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

NON-DEFECTIVE DEVICE

The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

SPARE PARTS

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

EXEMPT CLAUSES

Moretti does not offer any other declarations, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address Town/city

Sold to

Address Town/city

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Distribuito da / Distributed by:

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com