LEVITAS®

MA ST721-1e2_03 A_04-2013

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I Class Medical Device according to Directive 93/42/EEC and further modifications

ST721/1 Neoprene air cells cushion 1 section

44x39,5 cm - h=6,5 cm (inflated) - h=7,3 cm (deflated) - 8x7 cells

ST721/2 Neoprene air cells cushion 2 section

44x39,5 cm - h=6,5 cm (inflated) - h=7,3 cm (deflated) - 8x7 cells

1. Introduction

Thank you for purchasing a LEVITAS neoprene air cells cushion by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

2. Intended use

The LEVITAS by Moretti air cells cushion is conceived for the patient balance and for the protection against bed sores and tissue injuries. It guarantees an undifferentiated weight distribution, thanks to the pressure support in every position. The cushion follows the body of the patient and the ischemic parts can sink into it.

Who can use the air cells cushion?

- Everyone who has problems being seated
- For wheelchair users
- For anyone that need to stay seated for a long period (e.g. for the aftercare of an incident or an illness, elderly people, etc.)
- For anyone risking to develop pressure sores
- For anyone has to adjust pelvic obliquity

Why do we use air cells cushions?

- We use it to position properly the pelvic zone and the lower limbs. Pelvic zone and lower limbs are a solid base for trunk, head, neck and upper limbs
- For pressure relief
- For prevention of skin injuries

WE WOULD LIKE TO INFORM YOU THAT THE MOST EFFECTIVE WAY TO PREVENT THE DEVELOPMENT OF BED SORES IS TO CHANGE OFTEN POSITION, MOVING THE PATIENT FREQUENTLY

3. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the air cells cushions category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

4. General warnings

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the air cell cushion
- Keep the packed cushion away from heat sources
- SERVICE LIFE- the air cells cushion use limit is defined by the wear of the parts
- Pay attention to the presence of children

5. General description

The neoprene air cells cushion operates with an outward air micro-interchange (around 20cc/cm² per hour) which maintains the patient dry and guarantee him the best comfort. It guarantees a perfect weight distribution, exploiting to the utmost pressure relief in every position. It adapts itself to the individual shapes of the patient and allows the full immersion of the ischials.

The air cells cushion are:

- **ST721/1** 1 section
- **ST721/2** 2 sections

The kit is composed by:

- 1 neoprene air cells cushion
- 1 cover (**ST781**)
- 1 pump (**ST727**)
- 1 repairing kit
- 1 instruction manual

6. Before use

- Ensure the cushion doesn't have any damages which can compromise the operation
- Verify the cushion is correctly inflated
- Verify that any object is on the cushion before have seat

7. Using Warnings

- Always make reference to the manual before use
- Always pay attention to the presence of children
- Protect the cushion from sharp objects
- Avoid the exposure to the sun and to heat sources
- Cushion max loading weight is 200 kg. Do not overload
- Do not use the cushion if damaged
- Do not use devices for surfaces heating or cooling-off







8. How to use

The cushions are equipped with inflated pump

1. Open the pump ring counter clockwise and connect to the cushion . Close the pump ring turning it clockwise
2. Lift the lever situated on the pump head
3. Unscrew the valve on the top of the cushion (fig.1)
4. Inflate the cushion. Do not exceed with the inflation. If you use the manometer*, sit on the cushion and check the pressure reaches 30~50mmHg
5. Close the cushion valve
6. Press the lever and release the pump (if necessary open the pump ring). Repeat the process with other valves
7. Put the cushion into the cover. Cushion cover can protect your cushion to avoid it get dirty (fig.2)

8. Put the cushion on the wheelchair. Seat on the cushion check that pelvic zone is not in contact with the base (fig.3)
9. The patient is sank in the cushion for no more than 1-2cm. We suggest to put an hand between the patient and the cushion
10. Adjust the pressure in order to guarantee the best comfort. If necessary deflate by unscrewing the valve and let the air escape (fig.4)
11. Caution (fig.5)

*The manometer is an optional accessory (ST726)

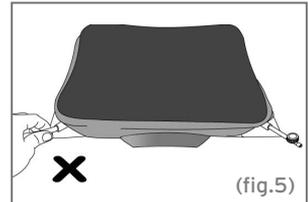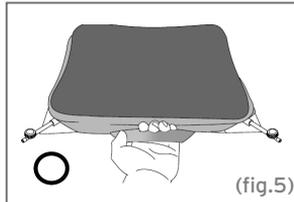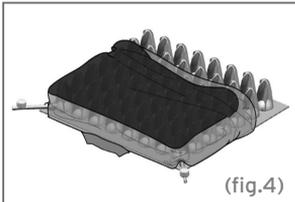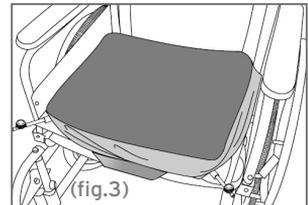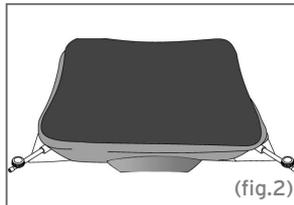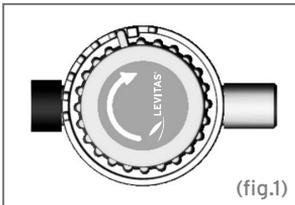8.1 Recommendation

Adjust cushion pressure with the pump every day

8.2 Suggestion

1. For the best use of the device, check the pressure of the cushion and verify it with the manometer scale (OPTIONAL). Be sure that the body is not in contact with the base of the cushion
2. Check cushion pressure if the environmental temperature or the altitude changed

8.3 Use of the cover

The air cushion is provided with a cover that protects the surface. The base of the cover is in nonslip material that blocks it on the chair and improves the balance of the user.

How to insert the cushion in the cover:

1. Lay the cover with the nonslip black base downward and the zip upward
2. Insert the cushion with the air cell side upward
3. Be sure that valves are placed outward (valves have corresponding holes in the cover)
4. Insert the cushion into the cover
5. Close the zip. The cushion is ready to use

9. Maintenance

The LEVITAS devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

10. How to repair the cushion

1. Deflate the cushion

2. Scratch the damaged part with sandpaper until it's smooth
3. Spread a thin layer of glue in the damaged zone and in the patch
4. Wait for 30 minutes. Press the patch down firmly until a good seal is achieved
5. Wait for 30 minutes. Make sure the sealed area is totally dried before inflating

11. Cleaning and disinfection

11.1 Cushion

1. Clean the cushion with water and mild soap
2. Wipe the cushion with a damp cloth and rinse out with cold water
3. Dry it

Warning!

- Use mild soap
- Do not clean in the washing-machine
- Do not use an autoclave to clean the cushion
- Keep the product far from sun rays

11.2 Cover

1. Clean in the washing-machine with cold water and soap
2. Dry it

Warning!

- Do not use drier, heater or bleacher
- Do not iron

12. Conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the air cushion in the appropriate disposal areas for recycling.

13. Spare parts/accessories

For spare parts and accessories refer to main catalogue

14. Technical features

MATERIAL	Neoprene
DIMENSION	44x39,5x6,5(h) cm (inflated) 44x39,5x7,3(h) cm (deflated)
MAX WEIGHT	200kg
STORAGE TEMPERATURE	0-40°C+10%
STORAGE HUMIDITY	10-90% RH+10%

15. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different

from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

15.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not covered by warranty: A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non-defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

15.2 Spare parts

The original Moretti spare parts are under warranty for six months. Accruing from the arrival.

15.3 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

 **MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN TAIWAN

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com