**LEVITAS®**

**ST900**

MA ST900 02\_A\_09-2012

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I Class Medical Device according to Directive 93/42/EEC and further modifications

## **ST900** Alternating pressure relief seat cushion

### **1. Introduction**

Thank you for purchasing LEVITAS cushion by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

In case of doubts, we recommend to contact the dealer who can help and advice you properly.

### **2. Intended use**

The cushion has been designed for the bed sore treatment and prevention, which may occur during long period seated on a wheelchair.

#### **CAUTION!**

- Do not use the product for a purpose not indicated in this manual
- Only qualified personnel or trained for the treatment and prevention of bed sore, can operate with the device
- Moretti S.p.A. declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

### **3. EC declaration of conformity**

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the antidecubitus cushions category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question ARE NOT MEASUREMENT DEVICES
3. The devices in question ARE NOT INTENDED FOR CLINICAL INVESTIGATION
4. The devices in question are sold in NON STERILE packing
5. The devices in question should be considered as belonging to class I
6. DO NOT use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

#### 4. Standards

The system has been tested and successfully approved to the following standards:

- EN 60601-1
- EN 60601-1-2

#### 5. General warnings

- Do not use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the device
- Keep the packed kit away from heat sources
- The device use limit is defined by the wear of the parts
- Do not allow children to play on or operate the cushion

#### 6. Electromagnetic compatibility warnings

This equipment has been tested and it complies with medical devices limits belonging to EN 60601-1-2:2007. These limits have been designed to provide reasonable protection against harmful interference in a typical medical installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guaranteed that the interference will not occur in a particular installation. If this equipment does cause harmful interference to other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference following one or more of the suggestions below:


- Reorient or relocate the receiving device
- Increase the separation between the equipment
- Connect the equipment into an outlet on a circuit different from that to which the other devices are connected
- Consult the manufacturer or field service technician for help

#### 7. Symbology


						
						
Code	Warning	Temperature limit		"BF" type device	Follow the instruction manual	Class II isolation
Production date	Device disposal according to EC/96/2002	ON	OFF	Manufacturer	LOT number	The device is protected against foreign objects 12,5 mm or more size.

**8. General description**


- 1. Unit pump
- 2. Cushion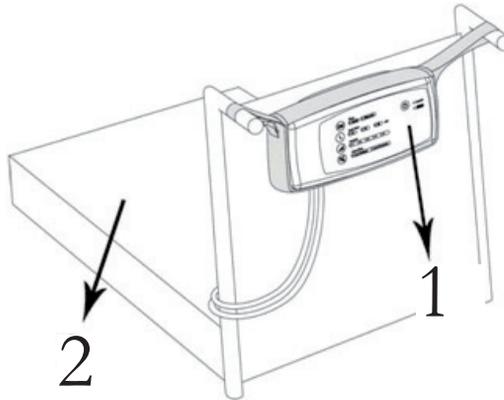

- 3. Front panel
- 4. AC/DC adapter connection
- 5. Air release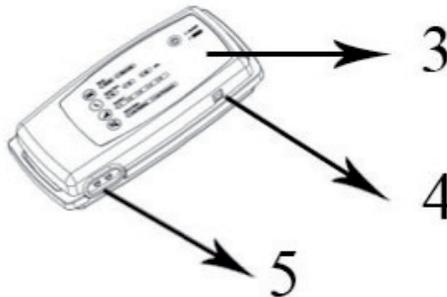

- 6. DC plug
- 7. AC/DC adapter

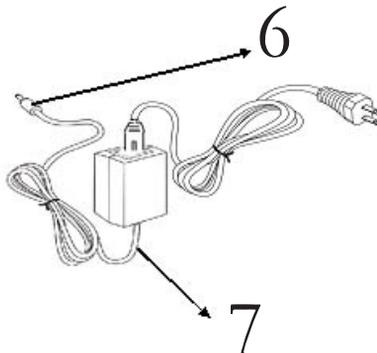## 9. Installation

Unpack the box and inspect for any damage which may have occurred during shipment. If there is any damages, please contact your dealer immediately.

1. Place the cushion on the wheelchair, the non-slip surface avoid any slide

**NOTE!** The pump must be used with a recommended cushion. Do not use for other purposes

2. Put the pump into its bag and hang it to the wheelchair handles
3. Connect the cushion to the pump

**NOTE!** Check and ensure the air hoses are not kinked or tucked under cushion

4. Connect the AC/DC adapter

**NOTE!** Use only original adapter and battery charger. Damages caused by other brand adapters are not cover from warranty

**NOTE!** The device works using batteries only if the electrical adapter is disconnected. Do not position the equipment so that it is difficult to disconnect the device

5. Push ON/OFF button to turn on the device. The led will be ON during the pump working

### 9.1 Battery

- Ensure the battery is completely charged (the led should be OFF). The first recharge is around 17 hours. the daily recharge time is around 12-14 hours
- When the DC plug is disconnected the device works with NI-MH batteries. When the pump is OFF the battery is still in stand-by and continue to use the charge. It takes 3 days to discharge the battery. We suggest to charge the battery during the night in order to use it during the day
- When the battery is exhausted the led start to flash. Recharge the battery or use the AC/DC adapter

**NOTE!** The pump will automatically turn off when the adapter is connected or disconnected during the operation

**NOTE!** If the battery is under charge for more than 24 hours, but the led is still on, disconnect the adapter for safety reason

### 9.2 State of the battery

Periodically check the battery led to verify the state of the battery. The leds shows:

- Exhausted battery
- Battery under charge

**NOTE!** During the operation the exhausted battery led can flash, in particular this happens during the cushion pressurization

**NOTE!** When the low pressure led start flashing there is a malfunction of the pressure. If the problem is not solved within 60 seconds, the sound alarm is activated. The low pressure can be caused by a loss of connection between the tubes. Check the connection are duly assembled following the instruction

## 10. Before use

1. Ensure the power cable is not damaged
2. Ensure the cover is not damaged
3. Check the tubes are correctly installed and are not damaged

## 11. Using warnings

**DANGER** - To reduce risk of electrocution -

1. Do not place or store this product where it can fall or be pulled into a tub or sink
2. Do not place in or drop into water or other liquid
3. Do not reach for a product that has fallen into water. Unplug immediately

**WARNING** - To reduce risk of burns, electrocution, fine or injury to people -

1. Evaluate patients for entrapment risk according to facility protocol and monitor patient appropriately
2. Close supervision is necessary when this product is used on or near children. Electrical burns or choking accident may occur if a child swallow a small part detached from the device
3. Use this product only for its intended use as described in this manual. Do not use accessories not recommended by the manufacturer
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, or if it has been dropped, damaged, or dropped into water. Return the product to a service centre for examination and repair
5. Keep the cord away from heated surfaces
6. Never block the air openings of this product or place it on a soft surface, such as a bed or couch, where their openings may be blocked. Keep the air openings free of lint, hairs, and other similar particles
7. Never drop or insert any object into any opening or hose
8. Do not modify this equipment without authorization of the manufacturer
9. Cushion cover has passed skin sensitization and skin irritation test. However, if you suspect that you may have had or are having an allergic reaction, please consult a physician immediately
10. If there is a possibility of electro-magnetic interference with mobile phones, please increase the distance to 3,3m between devices or turn off the mobile phone

### 11.1 Warning for the battery

1. Keep the battery out of the reach of children
2. The battery is composed by NI-MH batteries, improper use may cause explosion
3. Dispose the battery following the local procedures

## 12. How to use

### 12.1 Functions description

Button	Function
	Turn on and off the pump
	Turn on and off the sound alarm
	Change the therapy: alternating or static way
	Adjust the alternating cycle time in 10, 15 or 20 minutes
	Adjust the comfort accordingly to the patient demand and to the physician instructions. You can choose 5 different levels. 1 is the softer, 5 is the hardest. The first setting is level 1
	The orange led flash when the battery has low energy. To continue using the device recharge the battery or use AC adapter
	Shows that the pressure is not yet ready
	The led shows the battery is under charge

### 12.2 Operating instruction

1. Press the POWER button on the display panel to start the device
2. Before cushion is fully inflated, user can change the comfort setting according to their preference or physician's recommendations
3. Adjust the comfort range according to their preference or physician's suggestion. The unit provides two operation modes for option, alternating and static pressure mode  
The elements are constantly inflated on the static mode; in the alternate mode the channels are continuously exchanged following the cycle set up
4. The pressure alternating cycle time is also adjustable 10, 15 or 20 minutes.
5. When an abnormal low pressure condition occurs, the low pressure indicator will firstly light up and flash. Check the connections
6. Choose "ALARM MUTE" for silent mode
7. If the low pressure condition is not solved in 30 seconds, the pump will initiate the audible alarm

### 12.3 Battery replacement

1. Disconnect the plug from main power

2. Remove the cover of the battery holder
3. Remove the battery by pulling the velcro
4. Replace the battery and close the velcro
5. Connect the DC plug to the device
6. Connect the adapter to the power point to recharge the battery

- Use only original battery. Warranty doesn't cover any damage caused by an improper use
- High temperatures decrease the battery life. It is recommended to store in dry environment and away from heat sources
- The velcro is intended to secure the battery into its compartment. Always check the velcro is correctly fixed before close the battery holder

### 13. Maintenance

The LEVITAS devices by moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every year. In case of repair, use only original spare parts and accessories.

### 14. Cleaning and disinfection

Unplug the pump before starting the cleaning procedure:

- Clean the cushion with mild soap
- Do not immerse or soak pump unit
- After cleaning, dry the cushion without direct sunlight exposure.

#### **WARNING!**

Unplug the pump before starting the cleaning procedure

- Check the connection tubes are not obstructed by dust or else. in this case use a soft swab to clean the tubes.

#### **Cleaning**

Use only a damp cloth with mild soap. Wipe before use

Do not use water higher than 30°

Do not use water or vapour jet washing machine

#### **Disinfection**

If a disinfection is requested use a common cleaners

**NOTE.** Never use acids, alkalis or solvents such as acetone

### 15. Conditions of disposal

#### 15.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the kit in the appropriate disposal areas for recycling.

### 15.2 Treatment of batteries (Directive 2002/96/EC)

At the end of its life, the product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment or at retailers that provide this service. By ensuring these batteries are disposed of correctly, you will help prevent potentially negative consequences for the environment and human health which could otherwise be caused by inappropriate waste handling of the battery. To remark the need to dispose of electrical equipment separately, the product is marked with crossed mobile waste bin

### 16. Spare parts/accessories

For spare parts and accessories refer to Moretti main catalogue

### 17. Troubleshooting

If the following solutions are not enough for the problems you are facing, please contact your dealer

Problem	Solution
Power is not on	<ul style="list-style-type: none"> <li>• Check if the plug is connected to mains</li> <li>• Check if the power cord is well connected to the pump</li> <li>• During battery use, check the adaptor is disconnected</li> </ul>
Alarm is ON (audible & visual)	<ul style="list-style-type: none"> <li>• Check if the connection between air tube connector to pump unit is tightly secured</li> <li>• Check if all tubing connections along cushion are secured</li> <li>• Check if there is any leakage on air cells</li> </ul>
Cushion becomes too firm. Pressure can't lower down	The pressure is too firm for a long period, release some air by disconnecting the air tube connector and change to your preferred setting
No air is produced from the air tube connector	This is normal since it is at alternating mode. Air outlets take some minutes to produce air during their pre-set cycle
Low pressure - No solution	<ul style="list-style-type: none"> <li>• Check every cells. select static mode in order to inflate all the cells. Then verify the cells are in good condition</li> <li>• Check the tubes for any damages</li> <li>• In case of air loose contact your dealer</li> <li>• Verify if the pump can produce air. The air flow should alternate every half cycle</li> </ul>
The battery can't be recharged	<ul style="list-style-type: none"> <li>• After a 12 hours full recharge the pump should work for about 8 hours. if not, replace the battery</li> <li>• If the battery led is still on after 12 hours, replace the battery</li> </ul>

### 18. Electromagnetic declaration of conformity

#### Guidance and manufacturer's declaration - electromagnetic emissions

This device ST900 is intended for use in the electromagnetic environment specified below. The user of this device ST900 should make sure it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - Guidance
RF emissions CISPR 11	Group 1	The device uses rf energy only for its internal function. Therefore, its rf emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations flicker emissions IEC 61000-3-3	Complies	

#### Guidance and manufacturer's declaration - Electromagnetic Immunity

This device is intended for use in the electromagnetic environment specified below. The user of this device should make sure it is used in such an environment.

Immunity test	IEC60601 Test Level	Compliance	Electromagnetic environment - guidance
Electrostatic discharge (ESD)	6kV Contact  ± 8kV Air	± 6kV Contact  ± 8kV Air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Electrical fast transient/burst IEC 61000-4-4	± 2kV For power supply line  ± 1kV For input / out line	± 2kV For power supply line  ± 1kV For input/out line	Mains power quality should be that of atypical commercial or hospital environment
Surge IEC61000-4-5	± 1 kV Line(s) to line(s)  ± 2 kV Line(s) to earth	± 1 kV Line(s) to line(s)	Mains power quality should be that of atypical commercial or hospital environment

Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	$<5\% u_T (>95\% \text{ dip in } u_T)$ for 0,5 cycle  $40\% u_T (60\% \text{ dip in } u_T)$ for 5 cycles  $70\% u_T (30\% \text{ dip in } u_T)$ for 25 cycles  $<5\% u_T (>95\% \text{ dip in } u_T)$ for 5 sec	$<5\% u_T (>95\% \text{ dip in } u_T)$ for 0,5 cycle  $40\% u_T (60\% \text{ dip in } u_T)$ for 5 cycles  $70\% u_T (30\% \text{ dip in } u_T)$ for 25 cycles  $<5\% u_T (>95\% \text{ dip in } u_T)$ for 5 sec	Mains power quality should be that of atypical commercial or hospital environment If the user of this device requires continued operation during power mains interruptions, it is recommended that the device be powered from an uninterruptible power supply or a battery
Power frequency (50/60hz) magnetic field IEC61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of atypical location in a typical commercial or hospital environment
<p style="text-align: center;"><b>N.B.</b> <math>u_T</math> is the A.C. mains voltage prior to the application of the test level</p>			

**Guidance and manufacturer's declaration - Electromagnetic immunity**

This device ST900 is intended for use in the electromagnetic environment specified below. The user of this device ST900 should make sure it is used in such an environment.

Immunity test	IEC60601 test level	Compliance	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3Vrms/Veff 150 KHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of this device, including cables, than there commended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance $d = 1.2 \sqrt{P}$ 150 KHz a 80 MHz $d = 1.2 \sqrt{P}$ 150 KHz a 80 MHz $d = 2.3 \sqrt{P}$ 80 MHz a 2,5g MHz Where P is the maximum output power rating of the transmitter in watts (w) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>c</sup> (NOTE 3), should be less than the compliance level in each frequency range <sup>d</sup> (NOTA 4 ). Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

*Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.*

*Note 2: These guidelines may not apply in all situations. electromagnetic propagation is affected by absorption and reflection from structures, objects and people.*

*Note 3: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, am and fm radio broadcast and tv broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed rf transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.*

*Note 4: Over the frequency range 150 KHz to 80 MHz, field strengths should be less than 3 v/m*

**Recommended separation distances between portable and mobile RF communications equipment and this device:**

This device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of this device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and this device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter (m)		
	150 KHz to 80MHz $d=1.2\sqrt{P}$	80 MHz to 800MHz $d=1.2\sqrt{P}$	800MHz to 2.5 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where (P) is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

*Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.*

*Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.*

## 19. Technical features

### 19.1 Pump unit

Features	Specification
Power supply	DC 12V, 1,25A by adaptor
Power consumption	12VDC, 850mA
Battery pack	6V NI-MH ( 200 Ah) recommended NEXcell, LSD-AAx5
Dimensions	25 x 12,2 x 5,7 cm
Cycle	10, 15 and 20 minutes
Weight	1 kg (including adaptor and battery)
Pressure range	5 different values
Environment temperature	Operation: +5°C / +40°C Storage: -10°C / +50°C Shipping: -10°C / +70°C
Environment humidity	Operation: 30%-75% non - condensing Storage: 10%-75% non - condensing Shipping: 10%-90% non - condensing
Atmospheric pressure	Operation 700-1013,25 hPa
Classification	Class II, type BF, IP22 (with carry bag) Applied part: air cushion not suitable for use in the presence of a flammable anesthetic mixture (no AP or APG protection)

### 19.2 Cushion

Features	Specification
Dimensions (without flaps)	43x43x10 (h) cm
Material	Nylon + PU
Elements number	4
Max weight	120 Kg
Weight	1,1 Kg

## 20. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 20.1 Repairing

**Warranty repair:** If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

**Repair not covered by warranty:** A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

**Non-defective device:** The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

### 20.2 Spare sparts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

### 20.3 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)

MADE IN TAIWAN

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)