## GARANZIA

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## WARRANTY

This product has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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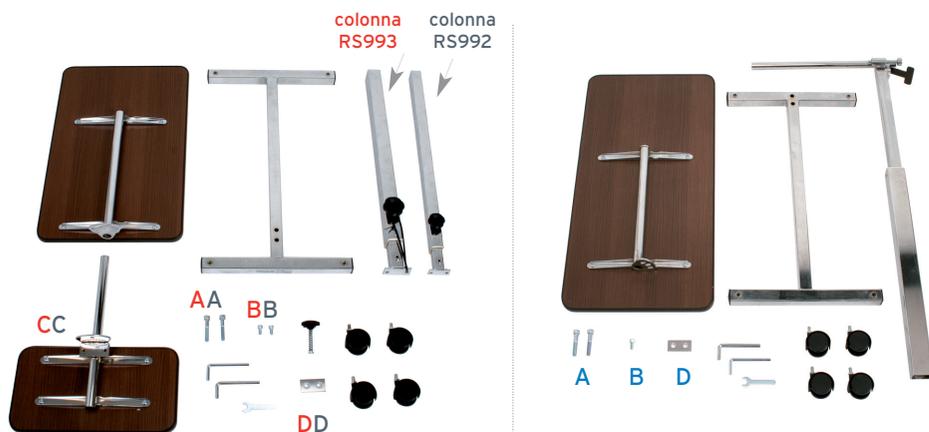
RS997M/N

### Codes RS992M/N\* - RS993M/N\* RS997M/2\*

I Class Medical Device  
according to EEC directive 93/42

Thank you for purchasing the over bed tables, **MOPEDIA** by Moretti. Their design and their quality guarantee to the user comfort, safety and reliability. The over beds tables "Mopedia by Moretti" are designed and realized to meet your requirements.

This instruction manual give you suggestion for a correct use of the aid and good safety advice. We recommend to read carefully this instruction manual before using the over bed table. In case of doubts about the use, please contact your dealer.

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Only for RS993

Warning! Do not remove the safety band before point "3"

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1. Assemble the four castors with the help of the key.

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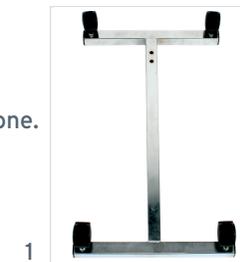
2. Assemble the column using the perforated plate (fig. D/D), the two big screws (fig. A/A) and lock them with the big Allen wrench.

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3. Only for RS993. Remove the safety band; the handle must be well tightened. Attention! The central bar is highly compressed and it could brutally come out.

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4. Assemble the small tray with the small screws (fig. B/B) using the Allen wrench given.

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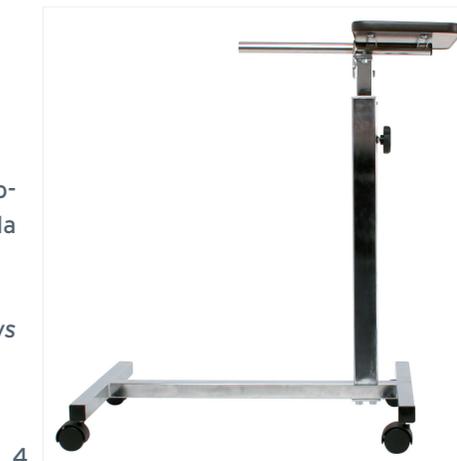
5. *To assemble the bigger tray insert it in the projecting tube of the small tray. Do not put them closer to each other.*

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6. *Insert the spring of the handles (fig. C/C) between two semicircles.*

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7. *Put the trays closer to each other, insert the locking screw and fix it with the handle.*

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## 8. Use

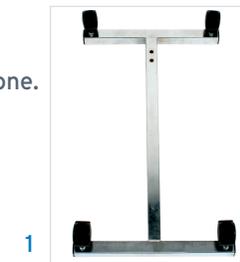
**Only for RS993. Turning the handle (E2) the tray will lift up automatically. Pushing on the tray, it will come down. Tighten the handle to block at the required height. Turning the handle (E1) the big tray can rotate. Tighten the handle again to block it at the required inclination.**

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1. *Assemble the four castors with the help of the key.*

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2. *Assemble the column using the perforated plate (fig. D), the two big screws (fig. A) and lock them with the big Allen wrench.*

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3. *Insert the tray in the projecting tube of the column.*

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5. *The bed tray is assembled. Turning the handle (C1) the tray will lift up automatically. Pushing on the tray, it will come down. Tighten the lever to block at the required height. Turning the handle (C2) the tray can rotate. Tighten the handle again to block it at the required inclination.*

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#### **Cleaning and maintenance for the overbed tables**

*Use a damp cloth to clean the tray from dust. Use water and soap to clean dirty parts.*

*To disinfect hospital spaces, follow the suggestion of a specialist on hygiene.*

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\* Overbed tables available in:

**Natural Colour** with codes:  
RS992N - RS993N - RS997N

**Brown color** with codes:  
RS992M - RS993M - RS997M

• RS992N

• RS993N

• RS997N