MOPEDIA

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I Class Medical Device
according to Directive 93/42/EEC and further modifications

RS909 - BATH BOARD**1. Introduction**

Thank you for purchasing a MopediA bath board by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

2. Intended use

The bath chair by Moretti is intended for disable patient assistance in bath tub

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. Ec declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the bath aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

4. General warnings

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- **SERVICE LIFE-** the product use limit is defined by the wear of the parts
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury
- **DO NOT** allow children to play on or operate the bath chair

5. General description

The seat is realized in non-slip plastic, with holes for water

6. Assembling

The bath chair is supplied disassembled. The package contains:

- Seat
- 4 supports
- Handle
- Screws

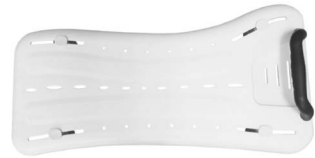
fig.1

To fit the handle, place the bath board with cleats facing downwards and introduce the handle to the holes at the end of the board. The handle grip should face towards the centre of the board (see fig. 1). From the underside of the Bathboard, screw the large headed white plastic screws into the holes at the ends of the handles. Make sure not to cross the threads. Tighten using a large headed screwdriver or a small coin.

6.1 - Fitting the bath board

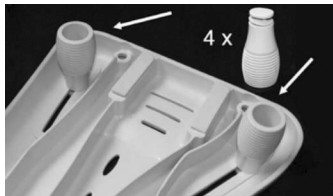
fig.2

1. Reverse the board and loosen the 4 supports, which hold the cleats in place. Make sure the cleats are free to move and that the non-slip white pads are facing the outside of the board (see fig. 2). Not suitable for straight sided baths
2. Slide the 4 cleats back towards the centre of the bath board so that they will easily fit down into the bath
3. Reverse the bath board so that the cleats hang down into the bath and the board is against the wall on the handle side. (Handle should always be on the wall side)
4. Position the bath board in the optimum position
5. Slide the cleats out so they touch the inside of the bath and tighten the wing screws finger tight (Do not over tighten)
6. Reverse the board again so that the cleats are pointing upwards and adjust them outwards by approximately 2mm on each cleat. Now tighten the supports so that they do not move easily
7. Turn the Bathboard up the right way and press it gently down into the bath (Do not force it). If the board slides along the bath, you may need to adjust the supports further outwards. To do this repeat steps 2 to 7

7. Before use

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

8. Using warnings

- Help the patient with movement difficulties to seat on the bath chair
- DO NOT use with patient who exceed the weight of 113 kg
- Avoid dangerous movement during patient positioning on the seat
- Pay attention to wet floors
- Periodically check the stability of the product, the screws are properly tightened and the rubber protections are in good conditions
- In case of doubts about use, adjustment or maintenance, contact your dealer

9. How to use

- Place the seat on the bath tub and tighten the supports in order to give the correct stability to the bath chair
- Place the handle on the opposite side of the entry side of the bath tub
- If necessary help the patient to seat on the bath chair
- Use the handle as aid for positioning and movements on the bath chair

10. Maintenance

The Mopedia devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair, use only original spare parts and accessories.

11. Cleaning and disinfection

11.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use

11.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

NOTE - Never use acids, alkalis or solvents such as acetone

12. Conditions of disposal

12.1 - General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

13. Technical features

13.1 - Dimensions

- Length: **75 cm**
- Width: **26 cm**
- Adjustable: **from 46 to 68 cm**
- Max loading weight: **113 kg**

14. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use.

Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty

period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not covered by warranty: A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non-Defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be paid by the customer.

14.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

14.3 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law.

Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

MOPEDIA

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

 MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com