MOPEDIA

MA RS813-14-16 01.A_06-2014

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I Class Medical Device
according to Directive 93/42/EEC and further modifications

1. Codes**RS813** Bath seat without backrest - width adjustable**RS814** Bath seat with backrest**RS816** Foldable shower chair**2. Introduction**

Thank you for purchasing a Mopedia bath board by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. Intended use

The bath chair by Moretti is intended for disable patient assistance in bath tub

**CAUTION!**

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the bath aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

5. General warnings

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- **SERVICE LIFE**- the product use limit is defined by the wear of the parts
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury
- **DO NOT** allow children to play on or operate the bath chair

6. General description

RS813-RS814: Bath seat with or without back, with steel frame and polyethylene seat. The model without backrest is adjustable in width for greater adaptability.

RS816: Shower chair with steel frame and polyethylene seat, folding to save space during storage. Non-slip rubber tips

7. Assembling

The RS813-RS814 products are supplied unassembled and consist of:

- 1 seat
- 1 right part of the structure
- 1 left part of the structure
- 1 back (only for model RS814)
- Screws for mounting

The RS816 comes mounted.

To use it:

1. Raise seat
2. Rotate the two sides on the front.
3. Lower the seat and fix it into the guides of the two sides

8. Before use

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

9. Cleaning and disinfection

9.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use

9.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner

Note: Never use acids, alkalis or solvents such as acetone

10. Conditions of disposal

10.1 - General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

11. Technical features

RS813

Dimensions

- Length: 50cm
- Width: 40 cm
- Seat: 40x40 cm
- Adjustable width: 50-63 cm
- Max weight: 110 kg

RS814

Dimensions

- Length: 50 cm
- Width: 40 cm
- Seat: 40x40 cm
- Backrest height: 36 cm
- Max weight: 110 kg

RS816

- Dimensions

- Length: 50cm
- Width: 50 cm
- Seat : 41 cm
- Max size: 53 cm
- Armrest height from the seat: 20 cm
- Seat height adjustment: 46-53 cm
- Max weight: 110 kg

12. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to wrong installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, not authorized alteration, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

12.1 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com