Moretti S.p.A.
Via Bruxelles 3 - Meleto
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

Fax +39 055 96 21 200
www.morettispa.com
info@morettispa.com

• RS936

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I Class Medical Device
according to EEC directive 93/42

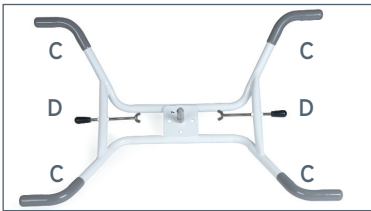
Code RS936

Thank you for purchasing the turning bath chair **MOPEDIA** by MORETTI, which has been manufactured according to the highest quality and safety standards and in accordance with the European Directive on Medical devices (93/42/EEC).

The frame is in white painted aluminium, the turning movement is in stainless steel, the seat is in plastic.

Assembly Instruction

The product is delivered in two parts : the base (A) and the seat (B).

A

B

Put the seat in the base and fix with the screw included (FIG. 1).

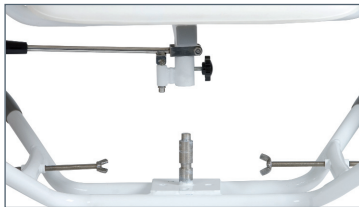
FIG. 1

Place the base on the bathtub and make sure that lateral mounts (C) are well put on the bathtub edge.

Adjust the lateral mounts (D) and make sure the seat is stable (FIG. 2.)

FIG. 2

To turn the bathchair use the lateral lever (E) near to the right armrest (FIG. 3).

FIG. 3

Use

Turn the bathchair in front of the patient.

Help patient to take correct place on the seat.

Turn again the bathchair to put the patient in the bathtub.

Cleaning

Mopedia turning bathchair is designed for minimum maintenance and requires the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Don't deep the parts in water. Drain the seat tubes after use and dry the seat thoroughly.

Warning

This product is recommended for people weighting kg 100 maximum.

The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.

- Check that the product has been delivered in perfect condition before you use it.
- In case of doubts about the use, adjustment or maintenance of the product, please contact your dealer.
- Worn, damaged or missing components must be replaced.
- Do not store the product outdoors.
- Periodically check the stability of the product, the screws are properly tightened and the rubber protection are in good conditions.
- Help people with movement difficulties to seat on the bathchair.
- Avoid any hard movement during patient positioning on the chair.
- Pay attention on wet floors, can cause problems during patient positioning on the chair.

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WARRANTY

This turning bath chair has 2-year warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get your turning bath chair repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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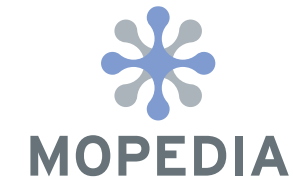
Acquistato in data / Purchasing date _____

Rivenditore / Authorized dealer _____

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Venduto a / Purchased by _____

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MORETTI S.p.A.
INTERNET: www.morettispa.com E-MAIL: info@morettispa.com
MADE in P.R.C.